

FCC Compliance CE


Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an output on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV Technician for help

This class B digital apparatus complies with Canadian ICES0003.

If device is in repeat mode, user's switch is being held down and a reset has occurred on the device, the user must release switch and press again to continue.

User's Manual

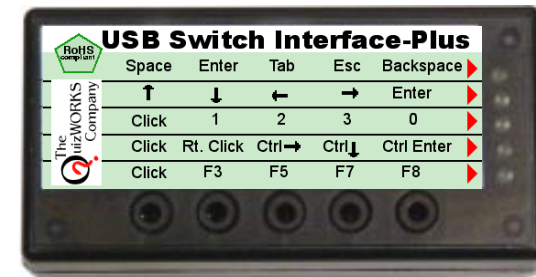

USB Switch Interface Plus

Product Features

- Supports 1 to 5 switches
- USB for Mac or PC/Windows
- Single or Repeat Mode
- Standard 1/8" plug for switch connection
- Supports industry standard single switch software applications
- 5 template configurations (LED indicates active template)
- Change templates with a touch of a button
- No additional USB software drivers required

The QuizWORKS Company

Mailing address:

P.O. Box 203
Ringwood, IL 60072-0203

Shipping address:

1395 Horizon Drive
Johnsburg, IL 60051-8420

888-526-1200 www.quizworks.com

Product Information

The USB Switch Interface Plus is the link between single switch input devices and Macintosh or PC/Windows computers with USB capabilities. The interface is utilized by software written for one to five single switches. Switches that have a standard 1/8" mini plug fit directly in the jacks on the interface. Switches with plug sizes other than 1/8" can also be used but an adapter is needed.

Installation



Connect the cable to the interface and computer. The USB Switch Interface Plus is ready when an LED lights up on the box.

Software


Driver software is included with Windows 2000 and newer and also is included on USB MACs OS 9 and newer.

First time activation for Windows OS

The first time you plug your USB Switch Interface Plus into a Windows OS computer, the correct driver will be installed on your computer.

Working With Templates and Jacks

There are five jacks and five LEDs on the interface. A lit LED indicates the active template, the jack your switch is plugged into determines the keystroke or mouse activity. For example, push the red button on the side of the interface box to the third LED, the LED is now lit. Plug a single switch into the 3rd jack from the left. Launch a text editor such as NotePad or Simple Text, press the single switch, notice the numeral 2 appears on your text editor window. To use single-switch software, just determine the keystrokes needed and select the appropriate template and begin operation.

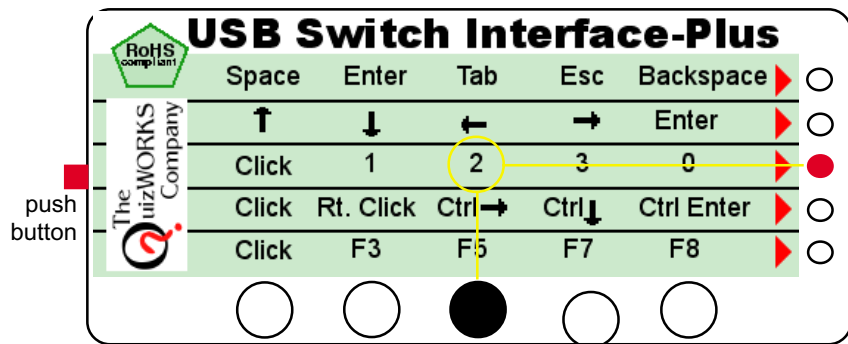

Label on USB Switch Interface Plus box with switch 3 and template 3 selected, will type a numeral 2 on a word processor when switch 3 is pressed.

Repeat Mode

This mode is used to repeat action when switch is held down. To turn ON press and hold red button on the interface until red LED flashes (about 15 sec) then release. When your switch is held down the red LED will flash and repeat action. Adjust repeat rate using your computer control panel. To turn OFF the repeat mode press and hold red button on the interface until red LED flashes (about 15 sec) then release.

Warranty or Repair Service

The USB Switch Interface Plus products are warranted against manufacturing defects for **one (1) year** from date of invoice. Warranty service covers materials and labor for the repair or replacement of the product. Problems caused by abuse, mishandling, improper installation, or improper storage are not covered by warranty. Any modification or repair attempt by the customer will invalidate the warranty.

We suggest you contact us at **888-526-1200** about any difficulties you may experience with The USB Switch Interface Plus. If the product needs to be returned for warranty or repair service, Customer Service will give you a **Return Authorization number (RA#)**. Ship the product under prepaid postage to:

The QuizWorks Company
Attn.: Repairs
1395 Horizon Drive
Johnsburg, IL 60051-8420

Please include with it :

- the **“RA” Number** given to you by the **Customer Service**
- a brief **description** of the problem
- your **return address**
- a **day-time phone** number or e-mail address
- a **proof of date of purchase** such as product registration card (on file @ CWI) a photocopy of the packing slip, invoice, purchase order or canceled check.

If it is a no-warranty repair over \$30, we will contact you with a cost estimate for your **“Approval to repair”**, Otherwise, we will repair the product and return it with an invoice product and return it with an invoice.

When repair service is required, ship the product to us under prepaid postage, whatever way is convenient for your, our goal is to return ship the product to you within 48 hours of receiving it , or within 48 hours of your **“Approval to Repair”**, if necessary.